DE	Deutsch	1
EN	English	2
FR	Français	3
IT	Italiano	4
ES	Español	5
UK	Українська	6
PL	Polski	7
NL	Nederlands	9
CZ	Čeština	10
SV	Svenska	11
HU	Magyar	12
BG	Български	13
DA	Dansk	14
PT	Português	15
SL	Slovenščina	16

DE

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Inhalt

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Instructions for use

Contents

Daily wear soft contact lenses immersed in isotonic saline solution

Intended use

Eye-sight correction.

Replacement Schedule

Refer to the packaging label.

Precautions

- 1) Follow the eye care professional's instructions for duration of use of the contact lens on a daily basis, follow-up visits, and emergency procedures.
- 2) Contact lenses should be removed immediately after contact with noxious vapour (e.g. chemical or hazardous substances) or hazardous environment with ocular impact.
- 3) Direct exposure of contact lenses to non-sterile water (e.g. tap water, whirlpool bath, swimming, participating in water sports) increases the risk of microbial infection.
- 4) The use of non-sterile water (e.g. tap water) in the handling of contact lenses and contact lens cases increases the risk of serious microbial infection.
- 5) Do not change lens type or parameters without consulting an eye care professional.
- 6) Do not use if the blister is damaged or seal is broken.
- 7) In the event of a side effect to the wearing of the contact lens, the user should immediately

remove the contact lens and consult an eye care professional before resuming use.

- 8) Aphakic patients should not be fitted with contact lenses until the determination is made that the eye has healed completely.

9) Different solutions cannot always be used together, and not all solutions are safe for use with all lenses. Use only recommended solutions.

10) Never use solutions recommended for conventional hard contact lenses only.

11) Chemical disinfection solutions should not be used with heat unless specifically indicated on product labeling for use in both heat and chemical disinfection.

12) If the lens sticks (stops moving) on the eye, follow the recommended directions on care for a sticking lens. The lens should move freely on the eye for the continued health of the eye. If nonmovement of the lens continues, the patient should be instructed to immediately consult his or her eye care practitioner.

13) Always wash and rinse hands before handling lenses. Do not get cosmetics, lotions, soaps, creams, deodorants or sprays in the eyes or on the lenses. It is best to put on lenses before putting on makeup. Water-base cosmetics are less likely to damage lenses than oil-base products.

14) Do not touch contact lenses with the fingers or hands if the hands are not free of foreign materials, as microscopic scratches of the lenses may occur, causing distorted vision and/or injury of the eye.

15) Carefully follow the for handling, insertion, removal, cleaning, disinfecting, storing and wearing instructions in the patient instructions for the contact lens and those prescribed by the eye care practitioner.

16) Never wear lenses beyond the period recommended by the eye care practitioner.

17) If aerosol products such as hair spray are used while wearing lenses, exercise caution and keep eyes closed until the spray has steered.

18) Always handle lenses carefully and avoid dropping them.

19) Ask the eye care practitioner about wearing lenses during sporting activities.

20) Inform the doctor (health care practitioner) about being a contact lens wearer.

21) Never use tweezers or other tools to remove lenses from the lens container unless specifically indicated for that use. Pour the lens into the hand.

22) Do not touch the lens with fingernails.

23) When wearing lenses that alter the eye color, temporary differences in the vision due to the presence of the color in the lens may be noticed especially in conditions of low light. If these differences in vision persist when wearing contact lenses, it is important to contact the eye care practitioner.

24) Do not change the contact lens care system without consulting an eye care professional.

25) Do not share contact lenses with others.

26) Remove your lenses before you go to bed.

27) Wearing the lens of the elders and toddlers must follow the directions of eye care practitioner.

28) If the power of lens of two eyes are different, should be careful wearing lens with left and right not to change.

Side effect

The patient should be informed that the following problems may occur:

- 1) Eye stinging, burning or itching (irritation) or other eye pain.
- 2) Comfort is less than when the lens was first placed on the eye.
- 3) Feeling that something is in the eye, such as a foreign body, or a scratched area.
- 4) Excessive watering (tearing) of the eyes.
- 5) Unusual eye secretions.
- 6) Redness of the eyes.
- 7) Reduced sharpness of vision (poor visual acuity).
- 8) Blurred vision, rainbow, or halos around objects.
- 9) Sensitivity to light (photophobia).
- 10) Dry eyes.

Warning

- 1) The tinted contact lens can reduce visibility in low light conditions.
- 2) The wearing of the contact lens while sleeping increases the risk of infection and permanent damage to vision.

Contraindications

Do not use contact lenses if any of the following conditions exist:

- 1) Acute and sub-acute inflammation or infection of the anterior chamber of the eye.
- 2) Any eye diseases, injury, or abnormality that affects the cornea, conjunctiva or eyelids.
- 3) Insufficiency of lacrimal secretion (dry eyes).
- 4) Corneal hypoesthesia (reduced corneal sensitivity), if not aphakic.
- 5) Any systemic disease that may affect the eye or be exaggerated by wearing contact lenses.
- 6) Allergic reactions of ocular surfaces or adnexa that may be induced or exaggerated by wearing contact lenses or use of contact lens solutions.
- 7) Allergy to any ingredient, such as mercury or thimerosal, in a solution which is to be used to care for the contact lens.
- 8) Any active corneal infection (bacterial, fungal, or viral).
- 9) If eyes become red or irritated.
- 10) The patient is unable to follow lens care regimen or unable to obtain assistance to do so.

Inserting the lens into your eyes

- 1) Rinse your lenses with multi-purpose solution before you put them on.
- 2) Put one of the lenses on the tip of your index finger. Make sure the lens is in a bowl shape; if not flip the lens inside out.
- 3) Hold the upper eyelid from above with one finger of your left hand.
- 4) Place the lens carefully on the middle of the cornea, holding the lower eyelid down with the middle of the right hand.

- 5) Slowly release upper eyelid, look down and gently close your eye.
- 6) Repeat the same procedure for the other eye.

Removing lens from your eyes

- 1) You may want to use a lubricating and rewetting drop just before you remove your lenses.
- 2) One or two drops in each eye will moisten the lenses and make them easier to remove.
- 3) Look up, holding your lower eyelid down. Pull the lens downward gently with your index finger and your thumb. Pinch the edge of the lens and remove it from your eye.
- 4) Repeat the same procedure for the other eye.

Disinfections

- 1) Before inserting the lens after removing it from the eye, always rinse the lens with contact lenses solution suggested by your practitioner.
- 2) After wearing your lenses, store them in a case filled with fresh contact lens solution and keep the storage case clean.
- 3) If you do not wear your lenses frequently, change the contact lens solution in the case with fresh solution once a week.
- 4) For removal of protein, do removal more than once a week by using protein remover or exclusive soaking solution which has a full removal function of protein.

Washing

- 1) Put the lens in the palm of the hand.
- 2) Drop the exclusive washing solution to 1-2 drops.
- 3) Rub lightly by turns to both sides of the lens.
- 4) Rinse cleanly with the saline.

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Dreamcon Co., Ltd.,
AR Experts B.V.