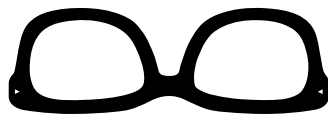
fielmann

CZ:

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EN:

We hope you enjoy wearing your new glasses!

In the following, we have put together some important information about your glasses along with a few tips on how to look after and store them properly.

Information about your sunglasses or blue-light filter glasses

Please note that sunglasses and blue-light filter glasses are regarded as personal protective equipment as per Regulation (EU) 2016/425.

In line with DIN EN ISO 12312-1, sunglasses are designed for general use and are intended to protect the eyes against UV radiation. This means that they are neither suitable for looking directly at the sun, nor as protection against artificial light sources (such as solaria), nor for use as eye protection against mechanical impact hazards.

Filter category DIN EN ISO 12312-1	Tint	Application
4	Very dark	Very high protection from extreme sun glare, such as on or around water, in snowfields, high mountains or deserts
Not suitable for driving or road use		
3*	Dark	High protection from sun glare
2*	Medium	Good protection from sun glare
1*	Light	Limited protection from sun glare
0	Very light	Very limited sun glare reduction
*Not suitable for driving at night or in low-light conditions		

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Glasses with a special blue-light filter reduce short-wave blue light emitted from digital devices, for example, and ease the strain on your eyes.

You can access the declaration of conformity for your sunglasses/blue-light filter glasses here: fielmann.de/declaration-of-conformity

Information about your reading glasses

Ready-to-wear reading glasses are a medical device as per Regulation (EU) 2017/745. All ready-to-wear Fielmann glasses meet the requirements of the DIN EN 14139 standard.

MD

- Only suitable for near vision and reading, and not for distance vision.
- Do not use them for driving.
- Do not use them as eye protection.
- Only regular, professional eye tests will ensure that you have the right eyewear and check the health of your eyes.

Not suitable for use when driving.

Tips for cleaning and storing your new glasses

Use a damp cloth and mild soap or eyeglass cleaner to clean the glasses, then dry them with a soft, clean cloth. Do not use solvents (e.g. alcohol, acetone) or aggressive detergents which may alter the features of the glasses. Dirty cleaning cloths may harm the filter properties. Please store your glasses so that the lenses do not get scratched – such as in a suitable case.

You can find all the Fielmann benefits on our website.

CH Importer/Importateur/Importatore:
Fielmann AG
Steinenvorstadt 62
4051 Basel, Schweiz

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