 Manufacturer: INTEROJO INC.
28&25,40 Sandan-ro 15beon-gil, Pyeongtaek-si, Gyeonggi-do, 17744, Korea
Contact Information: Seoul Office T) 02-780-1255, F) 02)761-6898

 EC Representative: Contact Lens Service GmbH
Oscar-Walcker-Strasse 26, 71636 Ludwigsburg, Germany

 Swiss Authorized Representative : MedEnvoy Switzerland
Gotthardstrasse 28, 8302 Zug, Switzerland

E) interejo@interejo.com www.interejo.com

Creation of Annex: July 2023

intended Use: Correction of eye sight

Intended users: Those who want to correct their eyesight and/or wear color contact lenses for cosmetic purposes

The product herein described is a medical device

How to Store before use: Stored at room temperature(1~30°C)

In the event of a serious incident related to the device, it should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

❖ Index

How to use/Caution/Adverse reaction/Warning

❖ Description

Common : This is indicated for the correction of refractive ametropia (myopia and hyperopia) in aphakic and/or not-aphakic persons with non-diseased eyes that may exhibit refractive. The soft contact lens is a daily wear contact lens. The lens is supplied in a sterile state, packaged in a phosphate-buffered saline solution. This device is a sterilized medical device by Moist heat (Steam) method of ISO17665-1. It is prohibited to use if the packaging is damaged before use and the sterilization barrier is collapsed.

Feature of the soft contact lens : Selective bluelight filter (Filters 16 ± 5% of bluelight in 400nm to 450nm wavelength)

❖ Lens Properties

The physical/chemical properties of the lens are:

1. Water Content : 55 ± 2%
2. Oxygen Permeability : 20.00 x 10⁻¹¹ (cm²/sec) · (mlO₂ /ml · mmHg) ± 20%
3. Main component : Methafilcon 45%, Water 55%
4. Saline : phosphate-buffered saline solution

❖ Lens Dimension (Only for those products.)

1. Optical zone : 6.00 ~ 12.00
2. Base curve : 8.10 ~ 9.00
3. Center thickness : 0.080 ~ 0.320
4. Diameter : 13.0 ~ 14.5
5. Diopter : Spherical Power: -12.00 ~ +10.00D

❖ How to use

1. Precaution

- ① Comply with the direction from eye-care practitioner and instruction strictly.
- ② Wash your hands before inserting and removing.
- ③ Be careful not to scratch lens with nails.
- ④ Check whether the lens is inside out or not.
- ⑤ Do not get cosmetic products, such as lotion, soap, deodorant or spray come in the lens.

2. Inserting and removing lens.

Inserting

- ① Wash your hands before inserting.
- ② Place the lens on your index finger. Hold your lower lid open with the middle finger of the same hand and hold upper eye lid open

with your left index finger of the other hand.

- ③ Looking straight at your eye in the mirror, slowly and gently place the lens on your eye.
- ④ Slowly take your fingers off when lens is placed correctly on your cornea.
- ⑤ Close and move your eyes to place the lenses in correct position.

Removal

- ① Wash your hands before removing.
- ② Check location of lens through mirror.
- ③ Look upside to move the lens downwards.
- ④ Hold the upper eye lid up with the index finger. With the other hand, hold the lower eye lid down with the middle finger and remove the lens by pinching the lower part of the lens with the thumb and index finger.

3. Use and Management

Washing	Put the lens on your hand and drop two or three drops of the cleaning solution, and washing the lens rubbing gently.
Rinsing	After washing, rinse the lenses with the managing solution, and put the lenses in their right or left side of the case with the storing solution.
Sterilizing (Chemical)	Fill the case with the storing solution and store it for the time which is specified by the manufacturer.
Preserving	Use the dedicated storing solution so that the soft lenses could not dry

Following indicated condition from instruction for use, it can be used regularly up to 30 days

4. Contact lens care

Overnight storage in saline solution results in contaminated lenses which if reused increases the infection risk especially with Staphylococci.

5. Purpose and effects of wearing lens

This product is lens that can be attached to cornea directly for vision correction.

When patient replaces lens every day, it can help reduce common illnesses, such as GPC(Giant Papillary Conjunctivitis) and improve slight discomfort and urtication related to patients allergy.

❖ Caution

1. Remember regular check-ups and always follow the advice of your eye-care practitioner.
2. When vision is not stable due to dryness, you can get normal vision by using eye lubricants and blinking.
3. When your eyes are tired due to fatigue, lack of sleep and reading long hours, remove lens and take a rest.
4. Do not get soap, shampoo, spray and cosmetic products in lens and put on makeup after inserting lens.
5. Use proper product for disinfection, rinsing or lubricant.
6. Do not use tap water and use professionally manufactured saline, cleansing and solution only.
7. Be careful not to scratch lens with nails.
8. Wash your hands before inserting lens.
9. Do not rub contact lens with paper or glass cleaning fabric to prevent scratch, pollution or damage.
10. When showering or swimming, remove the lens and use goggle.
11. We recommend patient to wear lens less than 8 hours a day.
12. Do not use when there is any damage to sterilized package.
13. Those who are exposed to strong acidity, weak basicity, dust or metal powder for a long time while working should refrain from using lenses.

❖ Adverse Reaction

When following any symptoms occurred, check the below action.

- eye irritation
- burning sensation
- stinging sensation
- too much tears secretion
- decreased visual acuity
- halo effect
- Dryness
- Bulbar redness
- Discomfort
- Blurry vision

[Action]

- Remove the lens. Even if the lens appears to be normal, do not use it again and follow the advice of eye-care practitioner.
- If lens is polluted, wear it again after cleaning. If same problem occurs, remove the lens immediately and follow the advice of eye-care practitioner.
- Contact lenses and blister lids must be discarded in wastebasket. The carton packaging and the blisters (polypropylene plastic) should be placed in the wastebasket or recycled in accordance with local waste guidance.

❖ Warning

1. Do not use lens if you have ocular disease or allergy to contact lens.
2. When you feel pain and irritation during wearing, remove the lens and get advice from eye-care practitioner.
3. Do not use eye drops while wearing lens. Use lens at least 4 hours later from using eye drops.

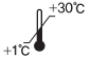
4. Do not use Damaged or expired lens.
5. Do not use lens that has been used by others.
6. Remove lens before sleep.
7. Pregnant women should follow advice of eye-care practitioner.

❖ Contraindications.

We do not recommend using contact lens if you have the following diseases.

- Infection in iris or cornea
- Other ocular diseases, wound or injury
- Irritation, pain, itchiness, serious or continuous dryness in eye
- Disease that can have effect on eyes by wearing lens
- Allergic reaction to contact lens
- Any systemic disease that may affect the eye or be exaggerated by wearing contact lenses
- Do not change the contact lens type or parameters before consulting your eye-care practitioner

Symbols key

Symbols key					
	Consult instruction for use	CAUTION	Sterile using steam or dry heat	USE BY	EU representative
					
Manufacturer	Batch code	Date of manufacture	CE Marking 1639	Temperature limit +30°C +1°C	UV Blocking
					
Unique Device Identification	Do not use if package is damaged and consult instructions for use	Medical Device	Single sterile barrier system with protective packaging outside	Country of manufacture KR	Single sterile barrier system
					
Swiss Authorized Representative					

 Hersteller: INTEROJO INC.
28&25,40 Sandan-ro 15beon-gil, Pyeongtaek-si, Gyeonggi-do, 17744, Korea
Kontaktinformationen: Seoul Office T) 02-780-1255, F) 02-761-6898

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Symbollegende

				
Consult instruction for use	CAUTION	Sterile using steam or dry heat	USE BY	EU representative
				
Manufacturer	Batch code	Date of manufacture	CE Marking 1639	Temperature limit +30°C +1°C
				
Unique Device Identification	Do not use if package is damaged and consult instructions for use	Medical Device	Single sterile barrier system with protective packaging outside	Country of manufacture KR
				
Swiss Authorized Representative	Single sterile barrier system			

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					
Consult instruction for use	CAUTION	Sterile using steam or dry heat	USE BY	EU representative	
					
Manufacturer	Batch code	Date of manufacture	CE Marking 1639	Temperature limit +1°C to +30°C	UV Blocking
					
Unique Device Identification	Do not use if package is damaged and consult instructions for use	Medical Device	Single sterile barrier system with protective packaging outside	Country of manufacture KR	Single sterile barrier system
					
Swiss Authorized Representative					

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